TRAVEL BATTERY INFORMATION

Battery Specifications: 25.2V 10.4 Ah (262.1 Wh)

Your Jazzy® Passport/iGo+® can travel up to 5.7 miles (9.2 km)¹ when using the Lithium-Ion travel battery. The Lithium-Ion travel battery has been tested and complies with United Nations Transport Regulations standard UN38.3 and conforms to 49 CFR 175.10(a)(17).

NOTE: Airline policies and regulations may vary. Please check with individual airline prior to travel.

NOTE: The battery charger is not included with the Lithium-Ion travel battery. The battery charger supplied with your Jazzy® Passport/iGo+® power chair must be used to recharge your Lithium-Ion travel battery.

¹ Varies with user weight, terrain type, battery charge, battery condition, motors, controller type, tire type, and tire condition. This specification is subject to a variance of +10%, -5%.

To prepare your Jazzy® Passport/iGo+® travel battery for airline transport:

1. Ensure that your travel battery is turned off (see figure 1-1) and your power chair is in the unfolded position.
2. Grasp the travel battery box handle and lift up the securement pin to disengage the box from the power base. See figure 1-2.
3. Slide the travel battery box out of the power base. See figure 1-3.
4. Insert the supplied terminal plug into the terminals on the back of the travel battery so they are not exposed. See figure 1-4.
5. Your travel battery is now ready for airline transport.

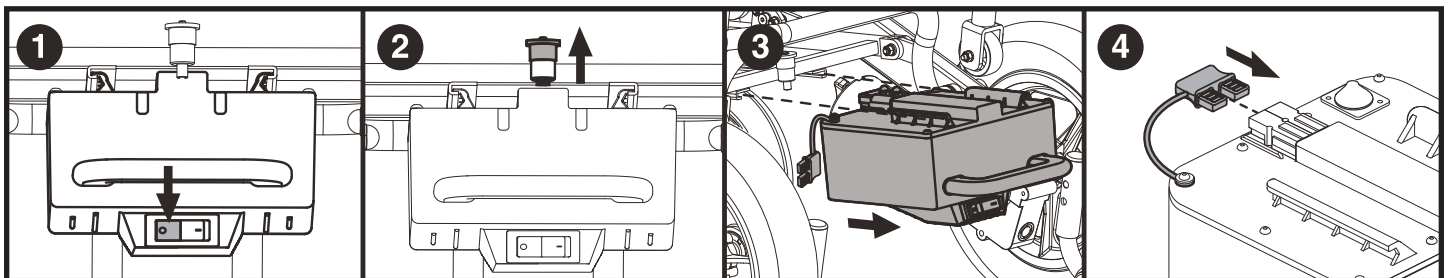
Figure 1. Transport Preparation

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