CHAIR WITH HEAT, CUPHOLDER, AND WIRELESS CHARGING

NOTE: The following information is in addition to the information in the owner's manual provided with your chair. It explains how to use the heat, cup holder, and wireless charging options, if equipped. You must read the owner's manual and all supplemental information before use. Please retain this information for your records, and contact your provider with any questions.

HAND CONTROL

The illustrations reflect those hand controls with the highest level of functionality. The hand control you receive may have fewer options, but basic functionality is the same.

HAND CONTROL IDENTIFICATION KEY

1. USB PORT

Use to connect to/charge USB enabled devices.

NOTE: iPhone users must use ONLY MFI-certified cables when charging iPhone devices.

2. HEAT CONTROL BUTTON (IF EQUIPPED)

Press the heat button to activate the heat function. Press the HEAT button again to cycle intensity through LOW, HIGH and OFF.

3. HEADREST IN BUTTON

Press and hold to move headrest in.

4. HEADREST OUT BUTTON

Press and hold to move headrest out.

5. LUMBAR SUPPORT IN BUTTON

Press and hold to move lumbar support in.

6. LUMBAR SUPPORT OUT BUTTON

Press and hold to move lumbar support out.

7. UP BUTTON

Press and hold to return all chair power options to the neutral position, while lifting you into an upright position.

8. DOWN BUTTON

Press and hold to lower chair to seated or reclined position.

9. SEATBACK RECLINE BUTTON

Press and hold to recline the seatback.

10. SEATBACK INCLINE BUTTON

Press and hold to raise the seatback.

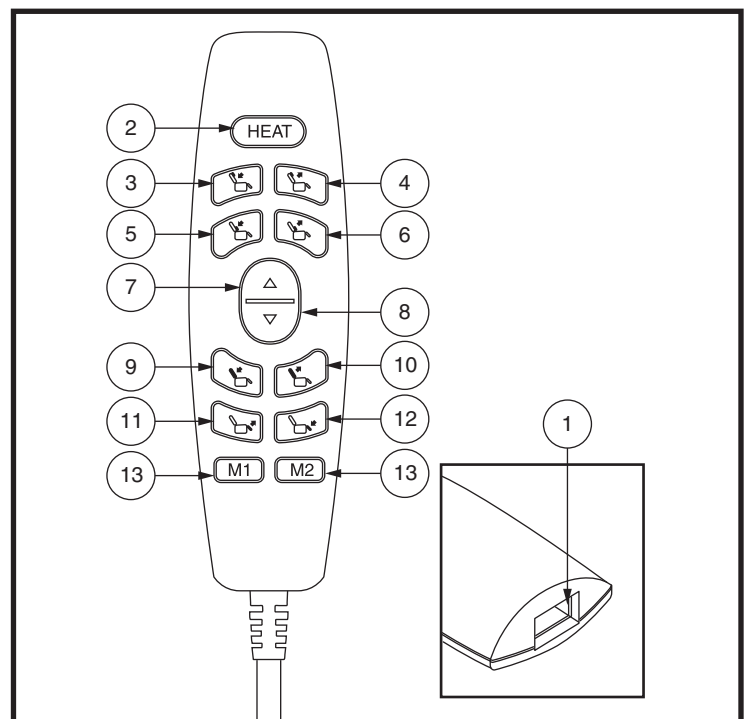
Figure 1. Hand Control

CHAIR WITH HEAT, CUPHOLDER, AND WIRELESS CHARGING, IF EQUIPPED

11. LEG REST UP BUTTON

Press and hold to raise the leg rest.

12. LEG REST DOWN BUTTON

Press and hold to lower the leg rest.

13. MEMORY BUTTONS (AVAILABLE ONLY ON SPECIFIC MODELS)

Press and hold Memory Button 1 and Memory Button 2. The LEDs on the memory buttons will flash to indicate that the hand control is in “set” mode. Press and hold the desired memory button to set the memory position. The LEDs will stop flashing. To go to a preset memory position, press and hold the desired memory button.

Follow these steps to properly stand up from your chair:

1. Activate and hold the UP button (switch) to raise your chair.
2. Release the button (switch) when the chair reaches a height where you can stand up comfortably.

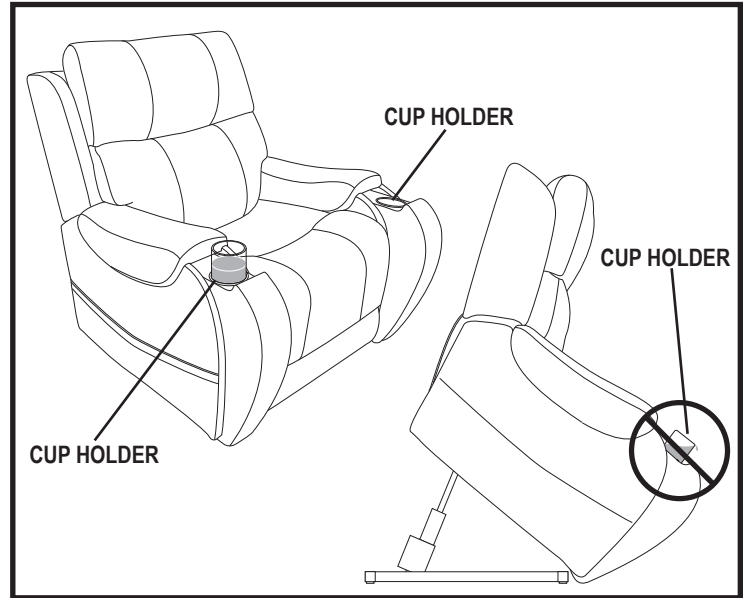
Figure 2. Cup Holders (if equipped)

WARNING! Do not attempt to stand up until you can stand safely, are steady on your feet, and can bear weight. We recommend the use of assistance aids and/or an attendant for enhanced stability.

WARNING! Avoid putting all of your weight on the chair armrests when attempting to stand and do not use the armrests for weight bearing purposes.

If your chair is equipped with cup holders, ensure the cup holders are empty prior to elevating your chair above a seated position to a standing position. See figure 2.

WARNING! Any contents in the cup holder may shift or spill when the chair is raised. Remove all contents prior to elevating your chair. Failure to remove all contents from the cup holders prior to elevating your chair above a seated position may result in personal injury or property damage.

WARNING! NEVER place lit tobacco products in the cup holders or attempt to use the cup holders as an ashtray.

WIRELESS CHARGER (IF EQUIPPED)

The wireless charger was developed for conveniently charging your smartphone. Ensure the screen of your smartphone faces up when it is placed on the charger as shown in figure 3.

NOTE: The wireless charger is only compatible with smartphones that have wireless charging capabilities. Additionally, no other smart devices, (i.e., smartwatches) should be used with the wireless charger.

NOTE: Your smartphone must lie flat on the charging surface. You may need to remove your phone case in order to properly position your smartphone for charging.

CHAIR WITH HEAT, CUPHOLDER, AND WIRELESS CHARGING, IF EQUIPPED

WARNING! Your smartphone may shift or fall out of the charger when the chair is raised. Remove your smartphone from the wireless charger prior to elevating your chair above a seated position. Failure to do so may result in personal injury and/or property damage.

WARNING! Do not leave your smartphone or any other item in the retractable arm when closing it. Failure to remove items before closing the retractable arm may result in personal injury and/or product damage.

WARNING! NEVER place any item other than a smartphone on the wireless charger. **NEVER** place any liquid, food, ashtrays, or lit tobacco products in the wireless charger. Doing so may result in personal injury or product damage.

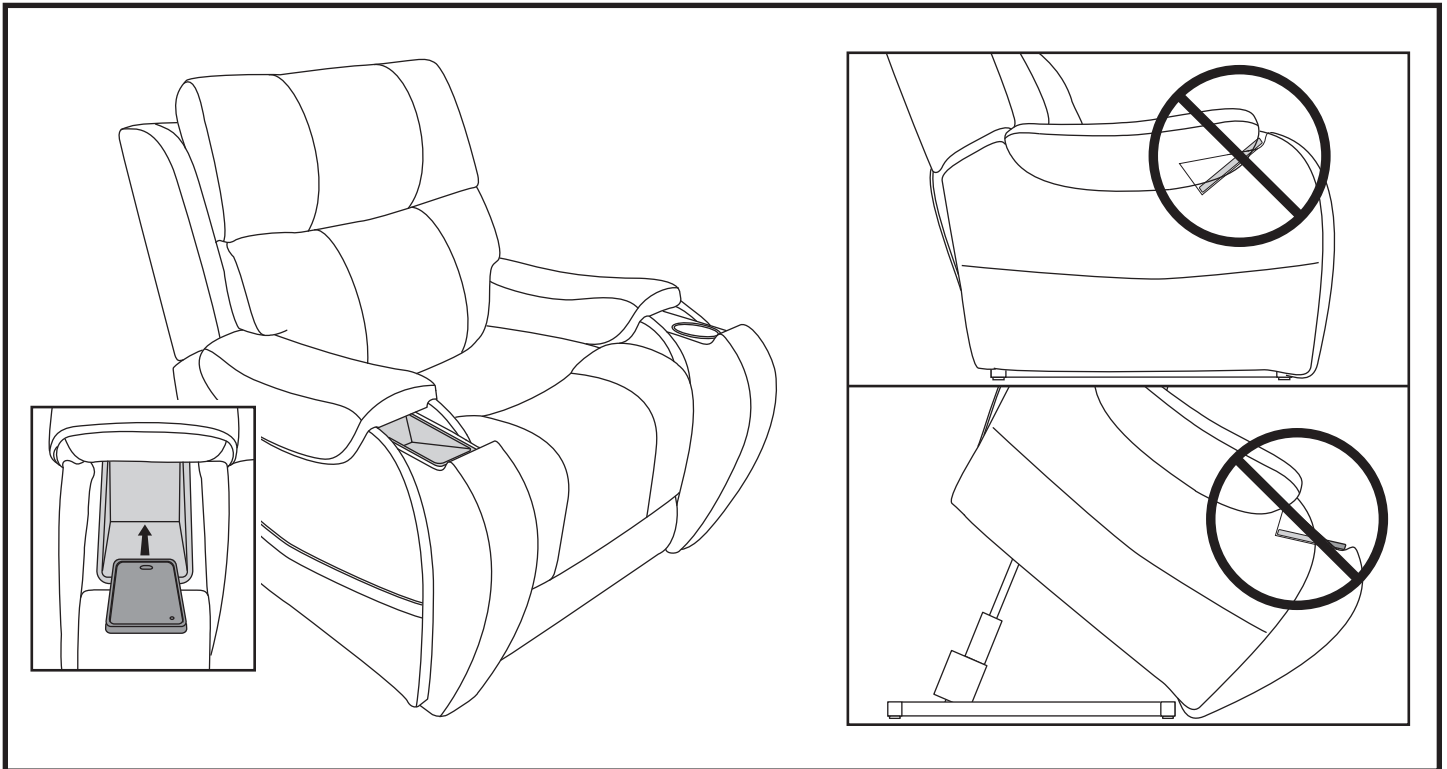
Figure 3. Charging your Smartphone on the Wireless Charger (if equipped)

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