**MV600 Suspension Replacement Instructions
Instructions de remplacement de la suspension MV600
MV600 instructies voor het vervangen van de ophanging
Anweisungen zum Austausch der MV600-Federung
Istruzioni per la sostituzione delle sospensioni MV600
Instrucciones de reemplazo de la suspensión MV600**

Tools needed:

- Standard socket set
- Standard wrench set
- #2 Phillips head screwdriver

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If you require any assistance with these instructions, please call **Pride Technical Service** at **1-877-800-1248**.
Canada providers, phone **1-888-570-1113**.

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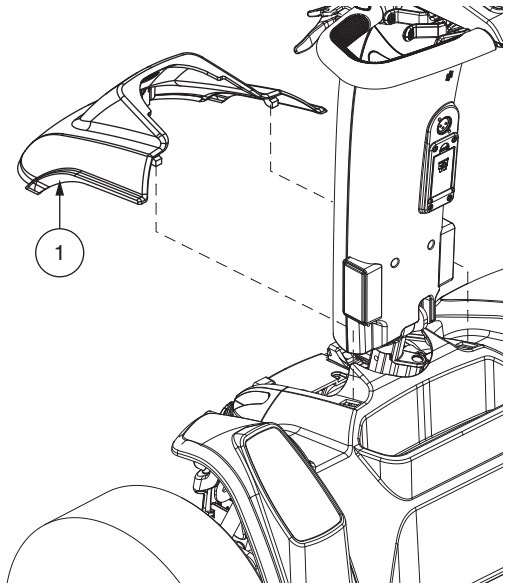
To replace front suspension:

Ensure your scooter is turned off and in drive mode, and set the scooter up on blocks. See owner's manual. Remove the hood shroud (1) from the front of the scooter as shown.

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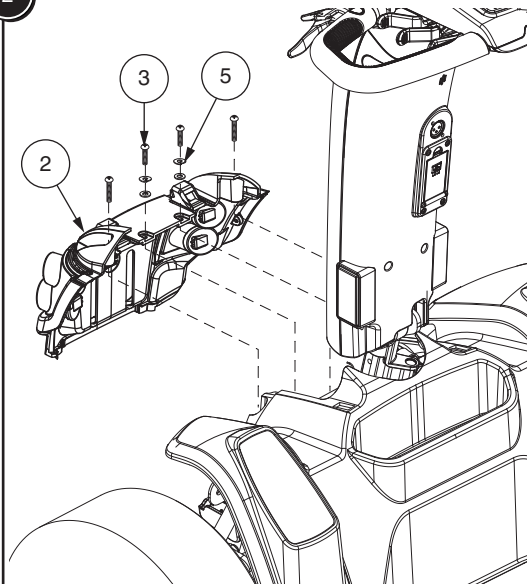
2

Uninstall the four screws (3) and four washers (5) from the grill shroud (2) and remove. Set all hardware aside.

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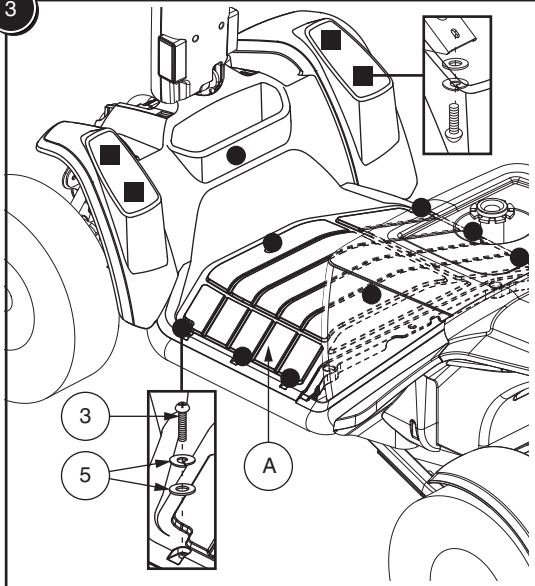
3

Uninstall the screws (3) and washers (5) that secure the main deck (A) to the scooter frame as shown. Remove the deck and retain all hardware.

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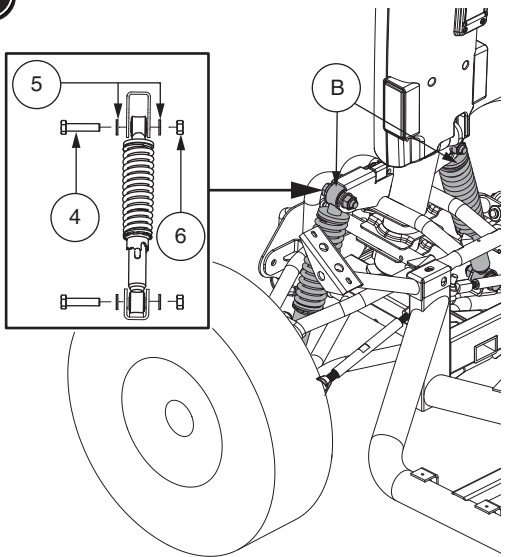
4

Locate the two shocks (B). Remove the the upper and lower bolts (4), four washers (5), and two nuts (6) to free the shock (B). Install the new shocks (B) and reinstall all hardware (4, 5, & 6).

Reinstall the main deck (A) and both shroud pieces (1 & 2) onto the front of the scooter. Your scooter is now ready for use.

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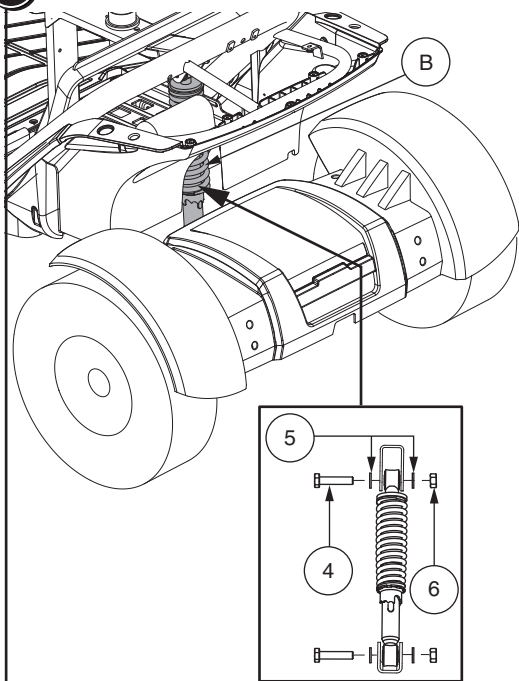
5

To replace rear suspension:

Ensure your scooter is turned off and in drive mode, and set the scooter up on blocks. Remove the seat and battery shroud. See owner's manual. Locate the rear shock (B). Remove the the upper and lower bolts (4), four washers (5), and two nuts (6) to free the shock (B). Install the new shock (B) and reinstall all hardware (4, 5, & 6).

Reinstall the battery shroud and seat. See owner's manual. Your scooter is now ready for use.

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